## AT HOME TEST FOR THE DETECTION OF PREGNANCY

EasyCup Pregnancy Test

### INSTRUCTIONS AND USE

#### INTENDED USE

The EasyCup Pregnancy Test is a rapid chromatographic immunoassay for qualitative detection of human chorionic gonadotropin (hCG) in urine in women who suspect they may be pregnant. The EasyCup Pregnancy Test is intended to detect the presence of the pregnancy hormone human Chorionic Gonadotropin (hCG) in women. **You can use the EasyCup Pregnancy Test on the day of your missed period or any day thereafter.**

#### WARNINGS AND PRECAUTIONS

**For in vitro diagnostic use only.** Do not use test components beyond the expiration date on the protective pouch. Make sure the protective pouch is sealed. Do not use if protective pouch seal is open, torn or damaged. Keep out of reach of children. For external use only. **Single-use device.** Do not reuse. Keep out of sunlight. Do not disassemble the cup and the test strip. Do not eat any component of this device. If accidentally swallowed, seek medical attention.

#### KIT CONTENTS:

One (1) cup sealed in a protective pouch with an integrated pregnancy test strip.  
One (1) moisture absorbent pack inside the pouch. The absorbent pack helps to preserve the test strip while enclosed inside the pouch. This pack is not to be used in connection with the test and should be discarded once the pouch is opened.

#### STORAGE:

Store the EasyCup Pregnancy Test at room temperature 38°F-86°F (4°C-30°C). The expiration date is marked on the protective pouch. Do not use past the expiration date.

#### Before you Begin:

CAREFULLY READ THE INSTRUCTIONS AND DO NOT OPEN THE FOIL POUCH UNTIL YOU ARE READY TO TEST. Failure to do so may result in invalid test results.

- Have a clock or timer.

#### Instructions (How to use):

- Remove the test cup from its protective pouch by tearing at the "notch". Use immediately.
- Open the test cup lid. The test strip is already inside of the cup. Do not remove the test strip.
- Urinate directly into the test cup. Fill at least to the minimum mark on the label. Do not overfill the cup with urine.
- Set the test cup on a level surface for 5 minutes. Do not shake the cup.

#### Wait

Wait **5 FULL MINUTES** before reading your results.

Pink lines will start to develop. If not, see "Invalid Result" below.

Test results should not be read after 15 minutes. The test results are invalid after 15 minutes.

#### Reading the Results:

Test results can be viewed through the cup. The test strip does not need to be removed from the cup to read the test result.

- If you see two lines you are pregnant. It does not matter how faint or dark (or how wide) the lines are.
- If you see only one pink colored line that means that you are not likely to be pregnant

**PREGNANT**  
If you see two pink lines you are pregnant.

**NOT PREGNANT**  
If you see only one pink line, you are likely not pregnant.

**INVALID RESULT**  
After performing the test (5 minutes) and no pink color line is visible within the control window, this result is considered invalid. The test has not worked, possibly because too much or too little urine was used. If no pink color line appears, the directions may not have been followed correctly. You will need to repeat the test by using a new EasyCup Pregnancy Test.

#### AFTER TESTING:

Flush the remaining urine sample in the cup down the toilet. Discard all components in a waste basket. Do not reuse the cup or test.

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### QUESTIONS AND ANSWERS

#### Q: When can I perform the test?

A: Test on the first day of your missed period and then any day after. You do not have to use first morning urine although it is highly recommended because if you are pregnant, the first morning urine generally contains the highest concentration of hCG.

#### Q: My result is Pregnant. What should I do?

A: Consult your doctor to confirm your pregnancy and determine what steps and precautions you should take next. Remember, early pregnancy care is very important to ensure the health of you and your baby.

#### Q: My result is not pregnant. What does this mean?

A: If your test shows a negative result, no pregnancy hormone (hCG) was detected in your urine. You may not be pregnant or you may have miscalculated your menstrual cycle and the pregnancy hormone level may not yet be high enough to be detected. If you are not sure of the result, you may want to re-test your urine within the first week of a missed cycle using a new test cup. If you still have a negative result and your period has not started, you should consult your doctor as there may be other reasons why you have not menstruated.

#### Q: How does the test work?

A: It detects a very small amount of the pregnancy hormone hCG (human Chorionic Gonadotropin) in your urine. hCG is produced in increasing amounts during the first trimester of pregnancy.

#### Q: When should I read my results?

A: The test results should be read as soon as possible after waiting 5 minutes. Test results should not be read after 15 minutes. The test results are invalid after 15 minutes.

#### Q: How accurate is the EasyCup Test?

A: Laboratory tests and consumer studies show the EasyCup Test to be more than 99% accurate. >99% accurate at detecting typical pregnancy hormone levels from the day of your missed period. Note hormone levels vary. However, some pregnant women may not have detectable amounts of the pregnancy hormone in their urine on the first day of the missed period or may have miscalculated the first day of their period.

#### Q: Will drinking alcohol or using oral contraception affect my test results?

A: Drinking alcohol will not affect your test results. Oral contraceptives, pain killers, antibiotics or hormone therapies should not affect the test results. Only medicines which contain hCG can produce a false result. Consult your doctor if you are not sure if your medicines contain hCG.

#### Q: Will the amount of liquid I drink affect the result?

A: Excessive fluid intake including water could dilute your urine, causing the amount of hCG present not to be detected properly. You should not drink too much liquid before testing.

### LIMITATIONS OF THE TEST

- This test is for in vitro diagnostic use only (not for internal use).
- Do not use this test past the expiration date.
- Make sure the package is sealed and do not open it until you are ready to use the test.
- Keep out of reach of children.
- Pain relievers, oral contraceptives, antibiotics and other commonly used medications (for example) should not interfere with the test.
- Certain health conditions, such as an ovarian cyst or ectopic pregnancy (pregnancy outside the uterus), can cause a false or irregular result.
- Procedures should be followed precisely for accurate results.
- All results should be confirmed by your healthcare provider, especially when making decisions about future medical care.

If you have any questions about the instructions or your results, call our Helpline Monday – Friday between 7:00 a.m. – 5:00 p.m. Eastern Time. Toll-Free 1-800-367-6022, or visit us at [www.FirstResponse.com](http://www.FirstResponse.com)

Manufactured for Church & Dwight Co., Inc. Ewing, NJ 08628  
FIRST RESPONSE is a trademark of Church & Dwight Co., Inc.  
[www.FirstResponse.com](http://www.FirstResponse.com)

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Fabricado por Church and Dwight Co., Inc. Ewing, NJ 08628.
FIRST RESPONSE es una marca comercial de Church & Dwight Co., Inc.